

danalogic GN

danalogic Ambio Smart

Your new Bluetooth
hearing aids



Your new danalogic Ambio Smart hearing aids have 'Bluetooth' capability.

This is great news as this feature gives you lots of benefits including being able to hear sounds from your phone (calls, music and audio) directly through your hearing aids.

This feature is available on the smart devices listed here:


<http://go.gn.com/app-support>

Advantages of Bluetooth hearing aids

- Phone calls whilst wearing hearing aids can be tricky. Your new hearing aids will allow you to listen to phone calls directly through your hearing aids, this makes it easier to hear what is being said. You can also listen to video calls in this way.
- Another benefit of your new hearing aids is listening to music, videos and any other audio directly through your hearing aids without having to wear headphones.

Connecting your hearing aids for Bluetooth streaming


If you have a compatible smart device, please follow the instructions in this booklet for either Apple or Android devices to connect your hearing aids to your smart device.

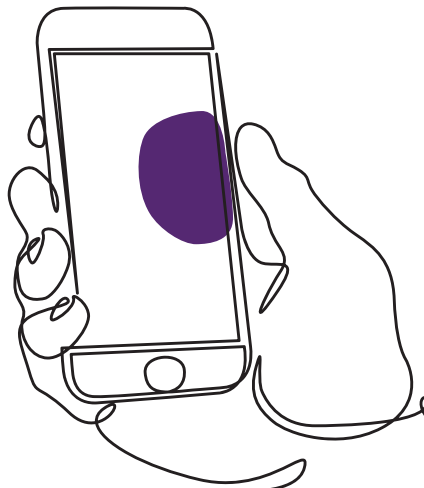


How to pair your hearing aids with your Apple device

Please follow these instructions in this specific order.

1. Turn your hearing aids off.
2. Turn on Bluetooth® by going to the Settings menu on your Apple device. The slider turns green when you activate Bluetooth.
3. Go back to the main Settings menu and then, tap on Accessibility.
4. Scroll down to Hearing Devices.
5. Turn your hearing aids on. Now, your Apple device will be able to detect your hearing aids.
6. Tap on Pair. If you have two hearing aids, you will need to accept twice.
7. Go to the App Store and search for 'BeMore' for your hearing aids.
8. Download the BeMore app, open it and tap Get Started.
9. Follow the instructions on the screen to connect to the app. You are now ready to go.

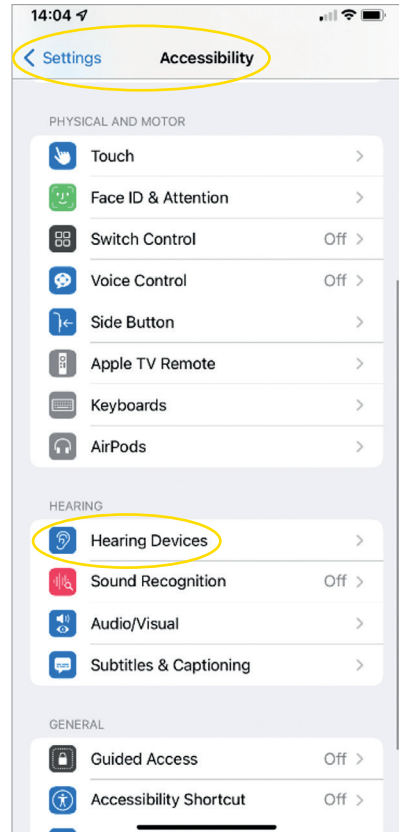

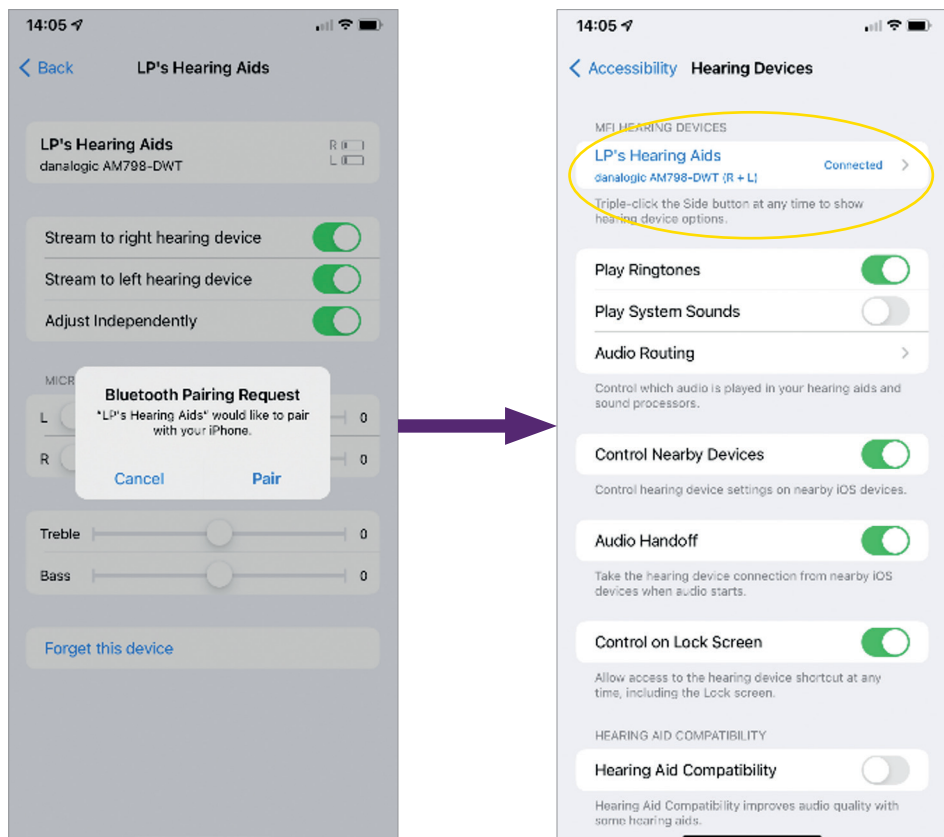


NOTE: These are the steps and how it looks in iOS 13 and newer

If you need to re-pair your Apple device


If one or both of your hearing aids have been replaced, you will need to re-pair them to your smart device.

1. Start by removing the old hearing aids by going to the Settings menu.
2. Scroll down and select Accessibility.
3. Select Hearing Devices.
4. Tap on your hearing aids, scroll down and tap on Forget this device.
5. Turn your new hearing aids off and on again and press pair. You will need to do this twice if you are wearing two hearing aids.

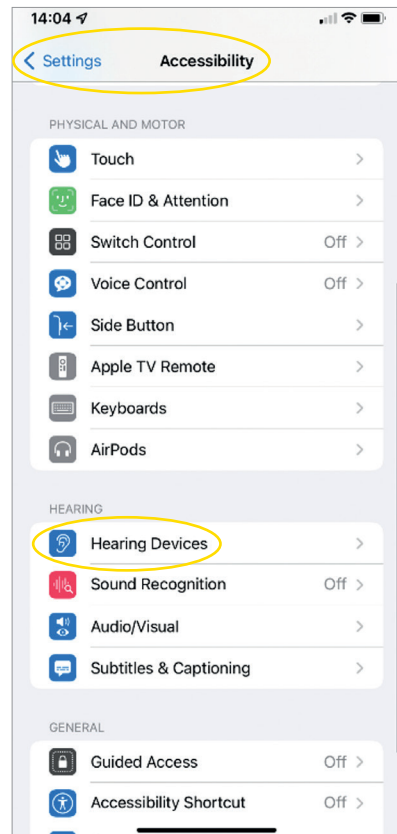

How to pair your hearing aids with your Android™ device

1. Go to the Google Play Store, search for BeMore and download the app.
2. Once downloaded, open the app and tap Get Started.
3. Follow the instructions on the screen to connect. You are now ready to go.

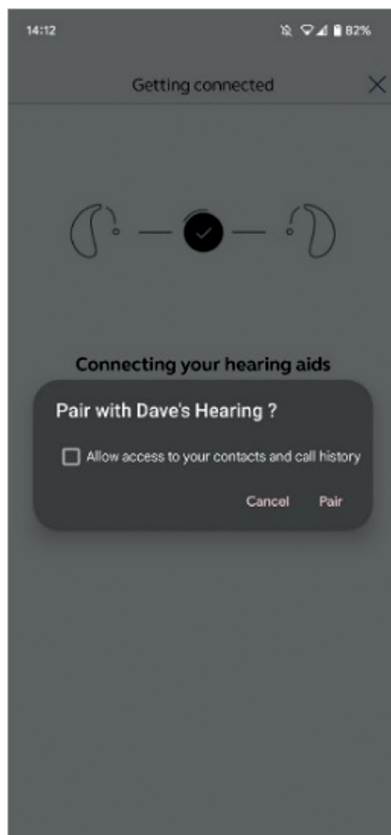

If you need to re-pair your Android device

If one or both of your hearing aids have been replaced, you will need to re-pair them to your smart device.

1. Swipe down from the top of the screen to open the Settings menu, then hold the Bluetooth icon for 2 seconds.
2. Hold the Settings icon for Hearing Aids for 2 seconds.
3. Select Forget.
4. Open the 'BeMore' menu.
5. Open the 'BeMore' menu and select 'Pair new hearing aids' then follow step 2 above.



danalogic GN

If you are having difficulties, please check our website **danalogic.co.uk/for-patients** for more guidance or contact your audiologist.



© 2021 GN Hearing A/S. All rights reserved. Danalogic is a trademark of GN Hearing A/S. Apple and the Apple logo are trademarks of Apple Inc., registered in the U.S. and other countries. App Store is a service mark of Apple Inc., registered in the U.S. and other countries. Android, Google Play and the Google Play logo are trademarks of Google LLC. The Bluetooth word mark and logos are registered trademarks owned by Bluetooth SIG, Inc.

[illegible]



danalogic GN

Danalogic from GN works harder to drive positive outcomes across the whole patient journey, enabling the NHS to deliver better care where it really matters.

Danalogic is part of the GN Group – pioneering great sound from world-leading hearing devices to Jabra office headsets and sports headphones. Founded in 1869 and employing over 5,000 people. GN makes life sound better.

Find out how you can help your patients get the most out of their hearing experience with Analogic from GN at **www.danalogic.co.uk**



GN Hearing UK Ltd.
Unit 13,
Talisman Business Centre,
Bicester OX26 6HR
Tel: +44 1869 352 800
Fax: +44 1869 343 466
danalogicuk@gnhearing.com
www.danalogic.co.uk